Aesculap[®] MIN Set

Instruments for Minimally Invasive Neurosurgery

Aesculap Neurosurgery

Aesculap[®] MIN Set Features

Fine instrument tips

Especially important when working in very small operating corridors and close to sensitive structures.

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Round golf ball handle design

Designed to ensure an excellent grip and enable easy rotation of the instruments. This provides precise handling.

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Various working lengths

One handle design aligned with precisely adapted working lengths. Always provides the right instrument at your fingertips!

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Slender design and angled bayonet shape

Improved visibility of the surgical site due to the slender design. Angled bayonet shape allows for less obstructions while working under the microscope.

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Noir[®], No Irritating Reflections

Aesthetic surface coating effectively prevents disturbing light reflections.

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Surgical procedures that reduce patient trauma have made minimally invasive procedures one of the main trends in neurosurgery. However, the protection of the close-by organ regions and prevention of functional deficiency remain a challenging task. For this reason, we have designed a specially dedicated set of fine instruments – the **Aesculap MIN set**.

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Aesculap[®] MIN Instruments

Micro Scissors | Mikro Scheren

Wavecut prevents tissue from slipping out of the jaws during cutting. The micro scissors are available in straight, slightly curved and curved.

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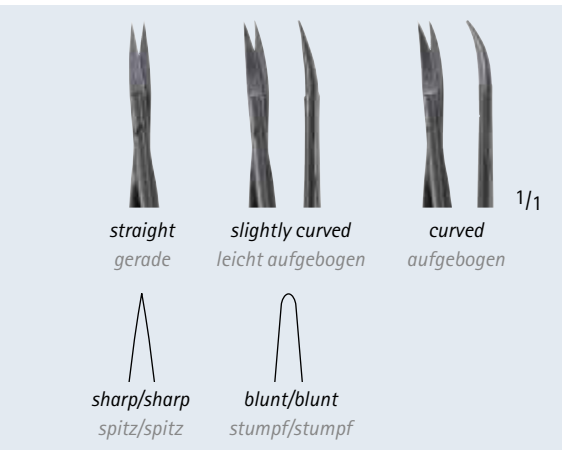
The fine cutting blades are available with sharp and blunt tips for both, sharp and blunt dissection. A pictogram on the spring indicates the working end.

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Feel the difference. The round ergonomic golf ball handle design enables an easy and uniform rotation of the instrument and offers an excellent grip to enable precise handling.

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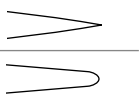
Aesculap[®] MIN Instruments

Micro Needle Holder | Mikro Nadelhalter

*Curved and smooth needle holder tips.
For suture material size 7/0 and smaller.*

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*For reliable grasping and handling of the needle, the micro
needle holders are equipped with ratchets.*

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*Feel the difference. The round ergonomic golf ball handle design
enables an easy and uniform rotation of the instrument and offers
an excellent grip to enable precise handling.*

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For suture material size
7/0 and smaller.

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The diamond dust coating ensures an even harder and more wear resistant surface on the inside of the jaws. Moreover, the structure of the DIA DUST® coating guarantees a reliable grip of the tissue.

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The micro tumor forceps have a serrated ring tip for quick and reliable tumor removal.

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
Aesculap[®] MIN Instruments

Micro Instruments | Mikro Instrumente

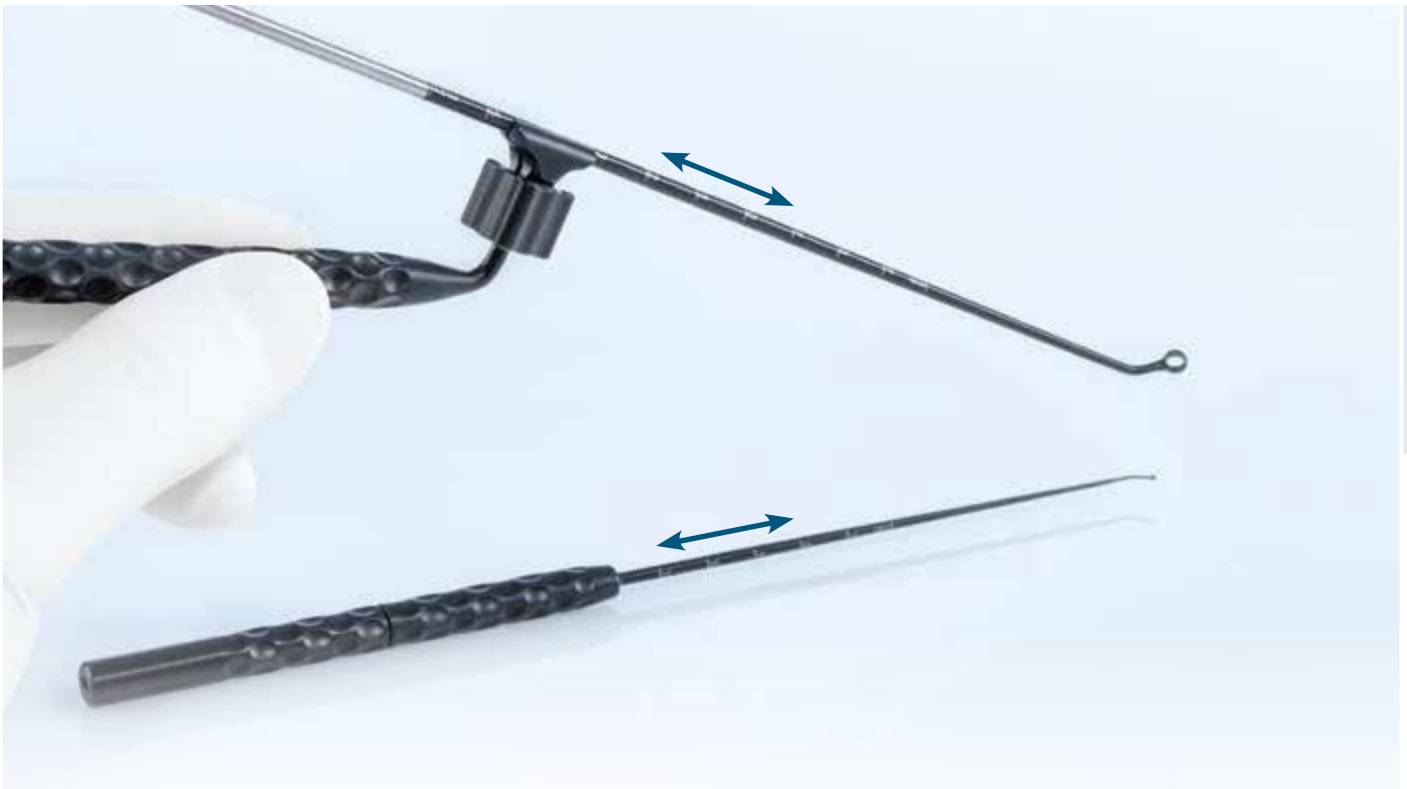The MIN Micro Instrument set is a modular system with interchangeable working ends. Choose between four different handle designs, two straight and two with an angled bayonet form, and 25 different working ends.

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The working length of the micro instruments can be easily adjusted according to the anatomical conditions. The handle allows for a stepless adjustment between 6 and 16 cm. Simply unscrew the handle, choose working length, fix handle, done.

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Rasparatory
Raspatorium

^{4/1}
Tumor knife
Tumormesser

In order for you to enjoy the high quality of your MIN Micro Instruments for a long time to come, they need good protection. For this purpose we have designed a specific storage tray to keep the instruments in place and enable easy use in the OR.

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Aesculap[®] MIN Instruments

Modular Suction Cannulas | Modulare Sauger

Modular suction tubes with atraumatic tips enable blunt dissection and retraction. The conical design of the tubes reduces plugging.

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Lateral bore holes reduce the suction pressure to enable suction close to sensitive structures.

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Length and diameter are clearly marked enabling quick and easy identification. Suction cannulas with lateral holes are additionally marked with a cirlet.

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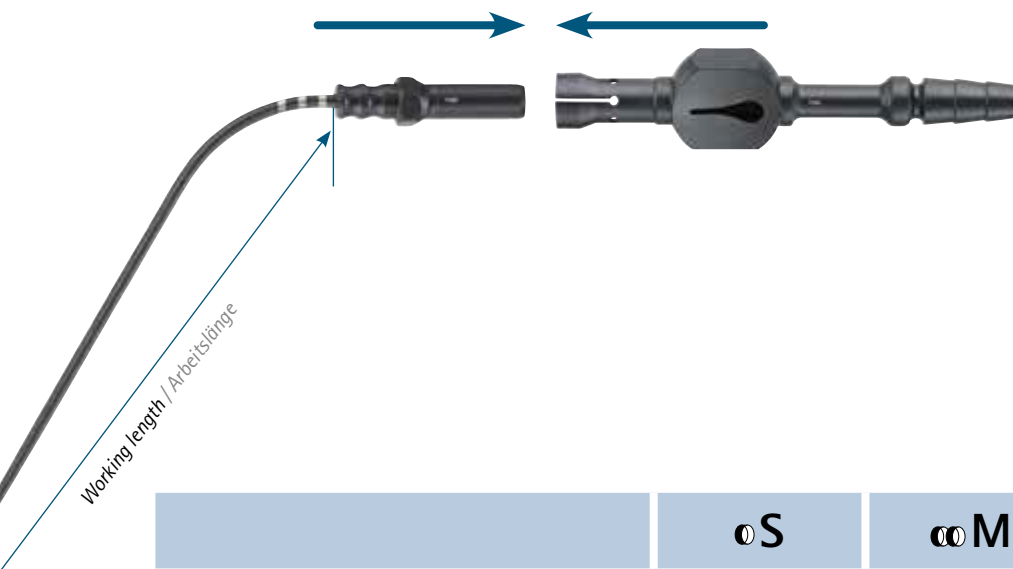
Greater cost efficiency and flexibility...

- Damaged suction tubes can be replaced individually
- Tray weight is reduced and less space is required
- Greater freedom for even more flexible use

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GF005B

Thumb control for modular suction tubes, tear drop shaped for a very precise suction regulation, incl. Mandrin.

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
Aesculap[®] MIN Instruments

Pivot-Point Bipolar Forceps | Pivot-Point Bipolare Pinzetten

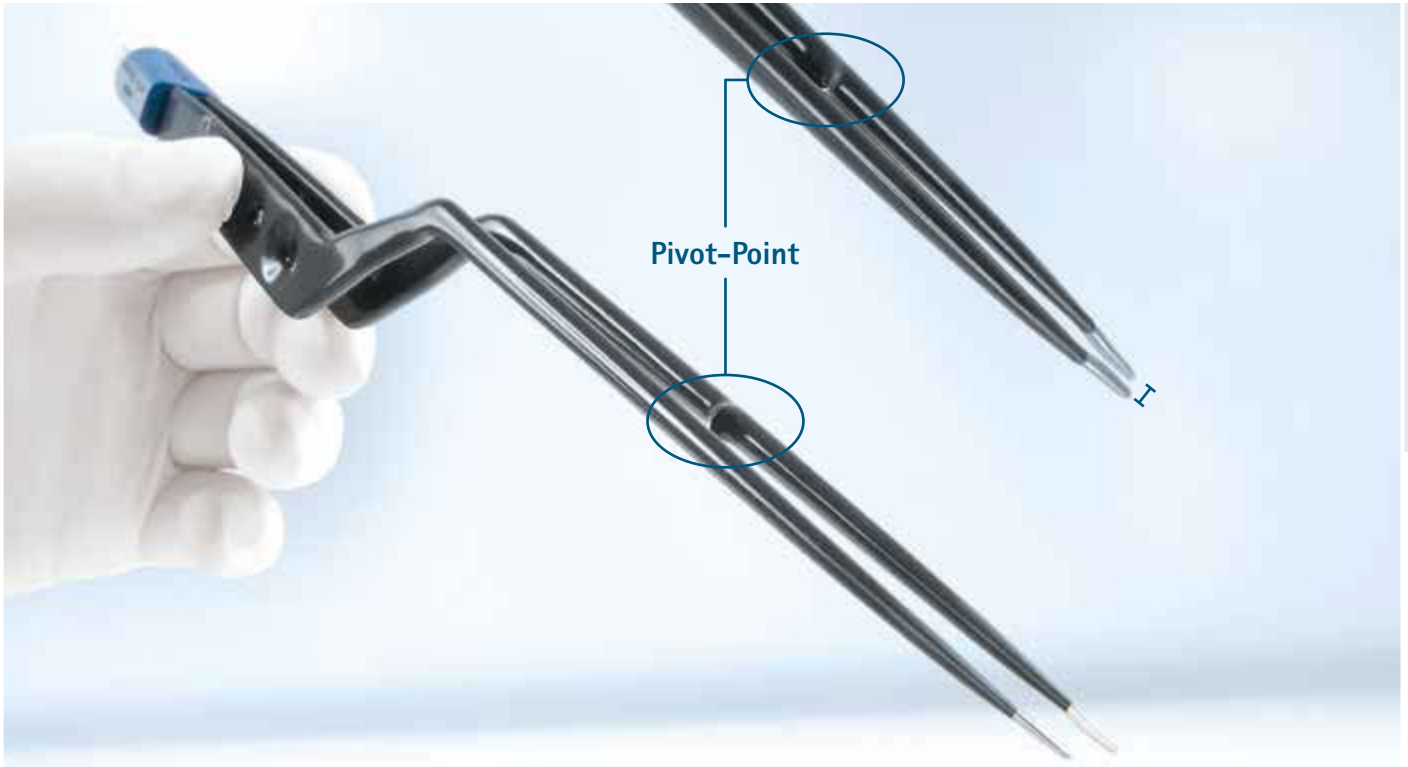The Pivot-Point eases coagulation when working in the deep seated and narrow surgical fields. Coagulating in these areas can be difficult as the forceps tips close and prevent current flow into the tissue. By applying additional force to the closed forceps, the Pivot-Point will reopen the tips and enable coagulation in these areas.

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The Pivot-Point bipolar forceps is designed with a Yasargil handle and especially slender working ends for deep operations and narrow approaches.

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The fine forceps tips are available in 0.7 mm and 1.0 mm in straight and curved. Upward curved tips offer improved visibility.

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| | | |
|--|---------------|---------------|
|  Aesculap tab | | |
| 0.7 mm  | GK822R | GK826R |
| 1.0 mm  | GK824R | GK828R |
| Working length | 95 mm | 135 mm |
| Total length | 215 mm | 255 mm |

| | | |
|--|---------------|---------------|
|  Aesculap tab | | |
| 0.7 mm  | GK823R | GK827R |
| 1.0 mm  | GK825R | GK829R |
| Working length | 95 mm | 135 mm |
| Total length | 215 mm | 255 mm |

Aesculap[®] MIN Set

Aesculap InOS[®] Trays | Aesculap InOS[®] Lagerungssiebkörbe

Fine tips need to be protected. With Aesculap InOS[®], that's no problem at all! This product's soft components, made from latex-free silicone, provide a soft cushion that keeps everything firmly in place.

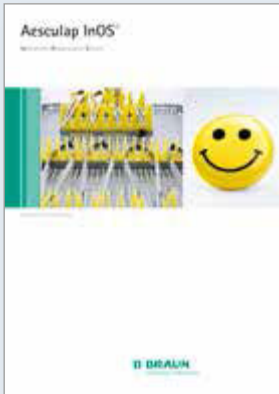
The accurately fitting instrument racks, mats with nubs and mesh mats for Aesculap instrument baskets provide ideal storage for these delicate instruments and hold them securely in place.

For the fine MIN Set instruments we have assembled two specially dedicated Aesculap InOS[®] trays, one for the MIN Micro Neuro Instruments, the other for the MIN Micro Instruments.

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For further information on the Aesculap **Instrument Organization System**, please refer to the brochure C81902.

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FD466R

Tray for MIN Micro-Neuro Instruments

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243 x 253 x 56 mm

Tray proposal for MIN Modular Suction Cannulas and Bipolar Forceps

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